EN	REFERENCE	
	KMINA FOLDING RAMP	K10072

The **KMINA FOLDING RAMP** has been designed to overcome obstacles, especially for transition between two types of grounds. It can also be used to bridge cables or other objects.

INSTRUCTIONS FOR USE

1. Unfold the ramp and place it so that it passes over the threshold where you want to locate it. The **folding mechanism** of the ramp should sit firmly flat and level on the threshold.
2. The **endings** of the ramp, with **rubber wedges** on both sides, should rest firmly on the level surface to ensure stability.
3. To transport the ramp, use the **handle** on the side of the ramp.
4. Make sure that the ramp does not move during use.

WARNINGS

- Remove and keep all packaging.
- Make sure that the ramp has not been damaged during transport or storage.
- Read the instructions and warning texts carefully before using the ramp.
- Keep the instructions for reference.
- Ensure that the surface at both levels is firm, stable and capable of supporting the intended load.
- The ramp is not designed for permanent outdoor use and must not be fixed.
- Any modifications/repairs that have not been carried out by the manufacturer will void the warranty.
- Check that the ramp is firmly positioned and stable each time it is used.
- No part of the wheelchair should catch and move the ramp.

CLEANING AND MAINTENANCE

- If the ramp is damaged or becomes slippery in any way, it must not be used (contact supplier/manufacturer).
- The ramp must not be cleaned with abrasive or corrosive products.
- Maximum recommended cleaning temperature 122F.
- **Includes case** for storage or transport.

FEATURES

WIDTH: 27.7"
OVERALL LENGTH: 35" - FOLDED LENGTH: 27.7"
CAPACITY: 600 lb - RAMP WEIGHT: 19 lb
HEIGHT: 1.6"

WARRANTY

IMPROVING MOBILITY DEVELOPMENT SL. guarantees that this device is free from defects. The warranty will remain valid for 3 years from the date of original purchase by the consumer. The warranty does not extend to parts exposed to natural wear and tear that require periodic replacement.

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