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Dear customer,

Thank you for placing your order. These shoes are a Class I Medical Device (MD), which complies with the requirements of Regulation (EU) 2017/745, while guaranteeing the safety and quality of the product with the CE marking.

Description

This footwear is designed and manufactured by **Salemedical®** in a specific last for footwear of special width, using technical high-quality materials for both the exterior and interior of the shoe, with bacteriostatic properties and memory foam. The sole of our shoe is made of high-strength and antistatic rubber in normal conditions, providing good grip when walking.

1. Intended Use

If you have any doubts, you should consult a healthcare professional. The **Salemedical®** products are designed for people with chronic systemic diseases and deformities that affect the feet:

- Skin conditions: Bacterial, viral and fungal infections.
- Type 1 and type 2 diabetes.
- Neurological diseases with sensory and motor involvement.
- Postoperative period: Splints, braces, support devices.
- Trauma-related conditions: Casts, splints, support devices.
- Rheumatic conditions: Arthritis or Arthrosis
- Circulatory disease: Oedema and swelling.

2. Place and conditions of use:

All of our models are specifically designed for the use that they are intended for, therefore, the use inside the home must be respected, the models for indoor use only must be used indoors, and the models for outdoor use must be dried after wear.

Please be aware of the circumstances, especially in wet or icy conditions, or when walking on snow or ice. In these cases, proceed with maximum caution.

3. Precautions

Check that the packaging contains a product identical to the image on the label and that it has not been damaged. If you detect any alteration of the product's properties or its performance, stop using it and contact the manufacturer as soon as possible.

4. Warnings

The use of **Salemedical®** footwear should be avoided in people with hypersensitivity to any of the constituent elements of the product. **Latex is contraindicated in people with latex allergy, as the sole of the shoe is made of natural latex or may contain traces of natural latex.**

5. Materials used:

Salemedical® models are made with different types of materials:

- Exterior fabric: different technical fabrics, depending on the model.
- Interior fabric or Lining: 100% polyester Coolmax / Carbon.
- Removable insole: 100% polyester Coolmax / Carbon over a base of synthetic rubber (LATEX).
- Outsole: Natural rubber (LATEX).

6. Proper use of the product

Before first use:

- Remove all packaging (paper, etc.) and adjust the shoes.

. If necessary, adjust the shoes to your foot length using the laces. For the use of the shoes, follow these instructions:

1. Sit on a stable support
2. Open the adjustable laces of the shoes
3. Place your foot inside the shoe
4. Close the shoe around your foot, without being too loose, but not too tight.