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LC166 - 12/2019

The Bluetooth® word mark and logos are registered trademarks owned by Bluetooth SIG, Inc. and any use of such marks by Laica S.p.A. is under licence.  
Other trademarks and trade name are those of their respective owners.

iOS, iPhone, iPad mini, iPod touch, App Store are trademarks of Apple Inc., registered in the U.S. and other countries - Android™, Play Store™ and Google Play™ are trademarks of Google Inc.

**LAICA**<sup>®</sup>

**LAICA S.p.A.**  
Viale del Lavoro, 10  
36048 Barbarano Mossano (VI) - Italy  
Tel. +39 0444.795314  
info@laica.com  
Made in China

**www.laica.it**

**LAICA**<sup>®</sup>

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**PS7030**

### PESABAMBINI ELETTRONICA SMART

pagina 4

**EN** Instructions and warranty page 7  
**SMART ELECTRONIC BABY SCALE**

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**SMART ELECTRONIC BABY SCALE - INSTRUCTIONS AND WARRANTY**

Dear customer, Laica would like to thank you for choosing one of our products, designed according to strict performance and quality criteria in order to ensure total satisfaction.

**IMPORTANT READ CAREFULLY BEFORE USE KEEP IN A SAFE PLACE FOR FUTURE REFERENCE**

The instruction manual must be considered an integral part of the product and must be kept for its entire life. If the appliance is transferred to another owner, its documentation must also be transferred in its entirety. To ensure the safe and correct use of the product, the user must carefully read the instructions and warnings contained in the manual insofar as they provide important information regarding safety, user and maintenance instructions. If the instruction manual is lost or in the event you require additional information or clarifications, please contact the company at the address shown below: Laica S.p.A. - Viale del Lavoro, 10 - 36048 Barbarano Mossano (VI) – Italy - Tel. +39 0444.795314 - [info@laica.com](mailto:info@laica.com) - [www.laica.it](http://www.laica.it)

Electronic baby scale that measures the weight of the newborn. Equipped with a tare function that allows you to weigh your baby using a soft cloth or towel and a LOCK-WEIGHT function, useful for showing the baby's weight on the display despite his or her continuous movements. Support bluetooth 4.0 and higher technology to automatically send the detected data to your smartphone, allowing you to monitor your child's growth at all times.

**SYMBOLS KEY**

Warning

Prohibition

**SAFETY WARNINGS**

- Before using, make sure the device is intact with no visible damage. If you have any doubts, do not use the device and contact the retailer.
- Keep the plastic bag used for packaging out of reach of children: suffocation hazard.
- This device must only be used for its intended purpose and in the manner indicated in the instructions. All other uses are considered improper and therefore hazardous. The manufacturer cannot be considered responsible for any damage due to improper or incorrect use.
- This device can be used by children on their own from the age of 8 upwards.
- Individuals with reduced physical, sensory or mental capacities or unskilled people must only use the product if previously instructed on its use, in safety and only if informed of the dangers connected to the product itself. Children must not play with the device. Cleaning and

maintenance operations must not be performed by children without adult supervision.

- In the event of a failure and/or malfunction, do not tamper with the device. Always contact your retailer for repairs.
- Handle the scale with care, protect it against knocks, extreme temperature fluctuations, humidity, dust, direct sunlight and sources of heat.
- ⊗ Do NOT allow liquids to penetrate into the body of the scale. This appliance is not waterproof. Do NOT leave and do NOT use the scale in water-saturated environments; humidity rates greater than 85% or contact with water or other liquids may cause corrosion or compromise proper scale operation.

**SAFE USE OF BATTERIES**

- Remove the batteries if the scale is not used for long periods of time and keep them in a well-ventilated and dry place, at room temperature.
- ⊗ Do NOT recharge the batteries if they are not rechargeable.
- ⊗ Do NOT recharge rechargeable batteries using methods other than those indicated in the manual or using unsuitable equipment.
- ⊗ NEVER expose the batteries to sources of heat or direct sunlight. Failure to observe these instructions may result in damage to and/or the explosion of the batteries.
- ⊗ Do NOT throw the batteries into an open flame.
- The removal or replacement of batteries must always be performed by an adult.
- Keep the batteries far from the reach of children: swallowing batteries carries the risk of death. If batteries are swallowed, immediately contact a doctor.
- The acid contained in the batteries is corrosive. Avoid contact with the skin, eyes or garments.

**PRODUCT DESCRIPTION (see fig.1)**

1. Display
2. Power "O/I" On/Off key
3. "UNIT" key to select unit of measurement
4. "TARE" key
5. Lock weight  key
6. Platform (or tray)
7. Battery compartment

**TECHNICAL SPECIFICATIONS**

- Measuring range: 30 g - 20 kg
- Division: 5 g
- 3 replaceable AAA 1.5 V alkaline batteries
- Backlit LCD display
- "Touch sensor" keys: pressure activation
- Operating frequency: 2402-2480 MHz, effective radiated power: <20 dBm

- Ambient operating conditions: +10°C +40°C; relative humidity ≤RH 85%
- Environmental storage conditions: +10°C +50°C; relative humidity ≤RH 85%

## INSTRUCTIONS FOR USE

### FIRST USE

- This scale works with 3 replaceable AAA 1.5V type alkaline batteries.
- To insert the batteries, open the battery compartment through the tab on the lid.
- Insert the batteries following the indicated polarity and close the lid.

### FUNCTION KEYS

#### O/I

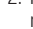
The "O/I" key is used for switching the scale on and off.

#### UNIT

Press this key to select the unit of measurement you prefer: kg or lbs. The capacity of this scale is 20 kg / 44.09 lb with division of 5 g / 0.011 lb. After turning the scale on, the display shows 0.000 kg / 0.000 lb.

#### (LOCK WEIGHT FUNCTION)

Due to the movements of the baby, the weight shown on the display varies continuously.

1. The scale automatically calculates the average of the weights perceived when the movements of the baby are slight. During the calculation on the display in the top left-hand corner the symbol  flashes, when it stops flashing, the weight of the baby is shown on the display.
2. It is possible to use the LOCK-WEIGHT function manually when the movements of the baby are more vigorous; press the  key: on the display the weight of the baby is displayed.

The weighed remains displayed on the display for a few seconds, then the scale switches off automatically.

#### TARE

The scale is equipped with the TARE function. When one prefers to lay the baby on a towel or soft cloth, proceed as follows:

1. Switch on the scale by pressing the "O/I" key for about 3 seconds. The display shows all the function symbols for a few seconds.
2. Wait for the display to show "0.000 kg / 0.000 lb", then place the towel on top of the tray.
3. Press the "TARE" key and wait for "0.000 kg / 0.000 lb" and the word "TARE" to appear on the display: the scale is ready to be used for weighing.

#### USE

1. Place the scale on a flat, sturdy surface.
2. Press the "O/I" key for about 3 seconds. When the display shows 0.000 kg / 0.000 lb the scale is ready to be used for weighing.
3. Place the baby on the tray of the scale and wait until the display shows the weight.

4. The scale automatically switches off if not used for about one minute. To extend battery life, turn off the scale by touching the "O/I" key for about 3 seconds.

Warning: do not overload the scale beyond its capacity, indicated on the bottom of the scale.

### OPERATION WITH APP

List of compatible devices:

- **iOS:** iPhone with 10 operating system.
- **For devices with Android™ operating system:** from version 6.0 (Marshmallow).

Support bluetooth 4.0 and higher technology, the scale sends the baby's weight to your smartphone, allowing you to monitor the baby's growth. During the following operations, we recommend keeping the smartphone near the scale to keep the bluetooth connection active.

1. Activate bluetooth in the smartphone settings.
2. Install the free "LAICA Home Wellness" app on your smartphone
3. Launch the App and follow the instructions to add the product and to create the new user by entering the child's details.
4. Turn on the scale by pressing the "O/I" key for about 3 seconds.
5. To send the scale-detected data to your smartphone, press the "DETECT WEIGHT" key in the app and then weigh the child.
6. The scale and your smartphone now display the weight of your child.
7. The scale switches off automatically after approximately 1 minute of non-use.

	<ul style="list-style-type: none"> <li>• During weighing, make sure that the app is active and displayed on your smartphone in order to verify that the data is sent immediately.</li> <li>• Before taking a new reading, always make sure that the bluetooth connection on your smartphone is active.</li> <li>• For Android™ devices it is necessary to activate the localisation services.</li> <li>• Before proceeding with a new weighing, always select the user where you want to store the data on your smartphone.</li> </ul>
---	--

### BATTERY REPLACEMENT

When the display shows "LO" or the battery symbol, proceed with the replacement of the batteries of the same type.

1. Open the battery compartment by pulling up the tab on the cover.
2. Extract the used batteries and dispose of them as indicated in the paragraph "Disposal procedure".
3. Insert the new batteries according to the indicated polarity.
4. Close the battery compartment.

## MAINTENANCE

- Store the appliance in a cool, dry place.
- Clean the scale with a damp cloth.

- Never use chemicals or abrasive products.

## TROUBLESHOOTING

Problem	Possible cause	Solution
The scale does not turn on	The batteries are depleted	Replace the batteries
The display shows "Lo" or the battery symbol	The batteries are empty	Replace batteries
The display shows "Err".	Weight above the capacity of the scale	Remove the weight from the scale so as not to damage it and check its maximum capacity
The "LAICA Home Wellness" app does not display any data.	The bluetooth function has not been enabled in the smartphone settings.	Enable bluetooth on your smartphone.
	The smartphone went in standby during weighing.	Keep the app active on your smartphone during weighing to check that data is sent instantly.
	The "DETECT WEIGHT" button has not been pressed in the app.	Press the "DETECT WEIGHT" button in the app to send the data from the scale to the app.

All weights are approximate, in that they are obviously rounded up or down to the closest division. We however guarantee an accuracy with tolerance within +/-1% + 1 division.

## DISPOSAL PROCEDURE

The symbol on the bottom of the device indicates the separated collection of electric and electronic equipment (**Dir. 2012/19/Eu-WEEE**).

At the end of life of the device, do not dispose it as mixed solid municipal waste, but dispose it referring to a specific collection centre located in your area or returning it to the distributor, when buying a new device of the same type to be used with the same functions. If the appliance to be disposed of is less than 25 cm, it can be returned to a retail location that is over 400 m<sup>2</sup> without having to purchase a new, similar device.

This procedure of separated collection of electric and electronic devices is carried out forecasting a European environmental policy aiming at safeguarding, protecting and improving environment quality, as well as avoiding potential effects on human health due to the presence of hazardous substances in such equipment or to an improper use of the same or of parts of the same. Caution! The wrong disposal of electric and electronic equipment may involve sanctions.

To correctly dispose of batteries (**Dir. 2013/56/Eu**) do not throw them into household waste but rather dispose of them as special waste at specialised recycling centres.

For more information about disposal of used batteries, contact the store where you purchased the

equipment containing the batteries, the municipality or the local waste disposal service.

This product is meant for domestic use. The conformity, certified by the CE-marking on the device, refers to the Directive 2014/30/Eu on electromagnetic compatibility.

This product is an electronic device that was tested to assure, according to the current technical knowledge, that it does not interfere with other nearby located equipment (electromagnetic compatibility) and that it is safe if used according to the advices given in the instructions for use. In case of operation anomalies, do not keep using the device and, if necessary, refer to the manufacturer directly.

The manufacturer, Laica S.p.A., declares that the device is compliant with Directive 2014/53/Eu. The complete statement of compliance is available at the following Internet site: [www.laica.it](http://www.laica.it).

## WARRANTY

This equipment is covered by a 2-year warranty starting from the purchase date, which has to be proven by means **of the stamp or signature of the dealer and by the fiscal payment receipt that shall be kept enclosed to this document.**

This warranty period complies with the laws in force and applies only in case the consumer is an individual. Laica products are designed for domestic use only; therefore, their use for commercial purposes is forbidden. The warranty covers only production defects and does not apply, if the damage is caused by accidental events, misuse, negligence or improper use of the product. Use the supplied accessories only; the use of other accessories invalidate the warranty cover. Do not open the equipment for any reason whatsoever; the opening or the tampering of the equipment, definitely invalidate the warranty. Warranty does not apply to parts subject to tear and wear along the use, as well as to batteries, when they are included. Warranty decays after 2 years from the purchase date.

In this case, technical service interventions will be carried out against payment. Information on technical service interventions, under warranty terms or against payment, may be requested by writing at [info@laica.com](mailto:info@laica.com). Repair or replacement of products covered by the warranty terms are free of charge. In case of faults, refer to the dealer; DO NOT send anything directly to LAICA.

All interventions covered by warranty (including product replacement or the replacement of any of its parts) shall not be meant to extend the duration of the original warranty period of the replaced product. The manufacturer cannot be made liable for any damages that may be caused, directly or indirectly, to people, things and pets resulting from the inobservance of the provisions contained in the suitable user manual, especially with reference to the instructions related to installation, use and maintenance of the equipment. Being constantly involved in improving its own products, Laica reserves the right to modify its product, totally or partially, with reference to any production need, without that this involves any responsibility whatsoever by Laica or its dealers.

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