**MOPEDIA®**

MA RI900 01 A\_ITA\_01-2022

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ENGLISH

## INDEX

<b>1. CODES .....</b>	<b>PAG.3</b>
<b>2. INTRODUCTION.....</b>	<b>PAG.3</b>
<b>3. INTENDED USE.....</b>	<b>PAG.3</b>
<b>4. DECLARATION OF CONFORMITY .....</b>	<b>PAG.3</b>
<b>5. GENERAL WARNINGS .....</b>	<b>PAG.4</b>
<b>6. SYMBOLS .....</b>	<b>PAG.4</b>
<b>7. GENERAL DESCRIPTION.....</b>	<b>PAG.5</b>
7.1 Description of the parts of RI900 .....	pag. 5
7.2 Description of the parts of RIA910 accessory arm.....	pag. 6
<b>8. ASSEMBLY INSTRUCTIONS .....</b>	<b>PAG.6</b>
8.1 RI900 assembly instructions.....	pag. 6
8.2 RIA910 accessory Assembly instructions .....	pag. 7
8.3 Assembly instructions for slings.....	pag. 7
<b>9. INSTRUCTIONS FOR USE .....</b>	<b>PAG.7</b>
9.1 Instructions for use of the RI900 patient lift .....	pag. 8
9.2 Instructions for use of RI900 patient lift with RIA910 lifting arm.....	pag. 8
9.3 Feature Details .....	pag. 9
9.4 Instructions for using the remote control .....	pag. 9
<b>10. WARNINGS.....</b>	<b>PAG.10</b>
<b>11. CLEANING .....</b>	<b>PAG.10</b>
<b>12. MAINTENANCE.....</b>	<b>PAG.10</b>
<b>13. SPARE PARTS AND ACCESSORIES.....</b>	<b>PAG.10</b>
<b>14. TECHNICAL FEATURES.....</b>	<b>PAG.11</b>
<b>15. CONDITIONS OF DISPOSAL.....</b>	<b>PAG.12</b>
15.1 Warnings for the proper disposal of the product as required by the European Directive 2012/19/UE .....	pag. 12
15.2 Treatment of dead batteries (Directive 2006/66/CE) .....	pag. 12
<b>16. WARRANTY .....</b>	<b>PAG.12</b>
16.1 Warranty for rechargeable batteries (if provided) .....	pag. 12
16.2 Reparations .....	pag. 13
16.3 Spare parts.....	pag. 13
16.4 Exoneration clauses .....	pag. 13

CE I Class Medical Device

REGULATION (EU) 2017/745 OF THE EUROPEAN PARLIAMENT AND OF THE COUNCIL  
of 5 April 2017 concerning medical devices

## 1. CODES

RI900 - Electrical lift Muevo Home - payload 135 Kg

## 2. INTRODUCTION

Thank you for choosing a patient lift and transfer device from the **MUEVO HOME** by **MOPEDIA** product range by MORETTI S.p.A.. Their design and quality are a guarantee of comfort, safety and reliability. **MUEVO HOME** by **MOPEDIA** patient lifts/stand up lifts by MORETTI S.p.A. are designed and built to meet all your demands for a practical, correct use. This user manual provides for some suggestions as to how correctly use the lift you have chosen and offers a lots of valuable advice for your safety. Please read carefully through the manual before using the patient lift/stand up lift. Should you have any queries, please contact your retailer for advice and assistance.

## 3. INTENDED USE

**MUEVO HOME** by **MOPEDIA** patient lifts are intended to lift and move people with motor difficulties. An assistant operates the machine. The lifts cannot be used by only one person. MORETTI S.p.A. refuses all liability for damages caused by improper use of the device or by its use for a purpose other than that one specified in this manual.

## 4. DECLARATION OF CONFORMITY

Moretti S.p.A. declares under its sole responsibility that the product made and traded by Moretti S.p.A. and belonging to the group of PATIENT LIFTS MOPEDIA complies with the provisions of the regulation 2017/745 on MEDICAL DEVICES of 5 April 2017.

For this purpose, Moretti S.p.A. guarantees and declares under its sole responsibility what follows:

1. The devices satisfy the requirements of general safety and performance requested by the Annex I of regulation 2017/745 as laid down by the Annex IV of the above mentioned regulation.
2. The devices ARE NOT MEASURING INSTRUMENTS.
3. The devices ARE NOT MADE FOR CLINICAL TESTS.
4. The devices are packed in NON-STERILE BOX.
5. The devices belong to class I in accordance with the provisions of Annex VIII of the above mentioned regulation
6. Moretti S.p.A. provides to the Competent Authorities the technical documentation to prove the conformity to the 2017/745 regulation, for at least 10 years from the last lot production

**Note:** Complete product codes, the manufacturer registration code (SRN), the UDI-DI code and any references to used regulations are included in the EU declaration of conformity that Moretti S.p.A. releases and makes available through its channels

## 5. GENERAL WARNINGS

### WARNING!

Always pay close attention to the presence of moving parts that could cause limb entrapment and injury.

- For proper use of the device refer to this manual, checking with a particular attention at the warnings before use.
- For proper use of the device to consult your doctor or therapist.
- Keep the packaged product away from any source of heat because the packaging is made of cardboard.
- The useful life of the accessory is determined by wear of parts that cannot be repaired and/or replaced.
- Be careful when children are nearby.
- Do not exceed the maximum payload.
- The user and/or the patient will have to report any serious accident that have occurred related the device to the manufacturer and appropriate authority of the State which the user and/or patient belongs to.

## 6. SYMBOLS

Product code

Unique Device Identification

CE mark

Manufacturer

Batch Lot

Read the instruction manual

Medical Device

Conditions of disposal

Attention, should read the instructions

Production dates

Waste Electrical & Electronic Equipment (WEEE)

Protection against dust and water splashes

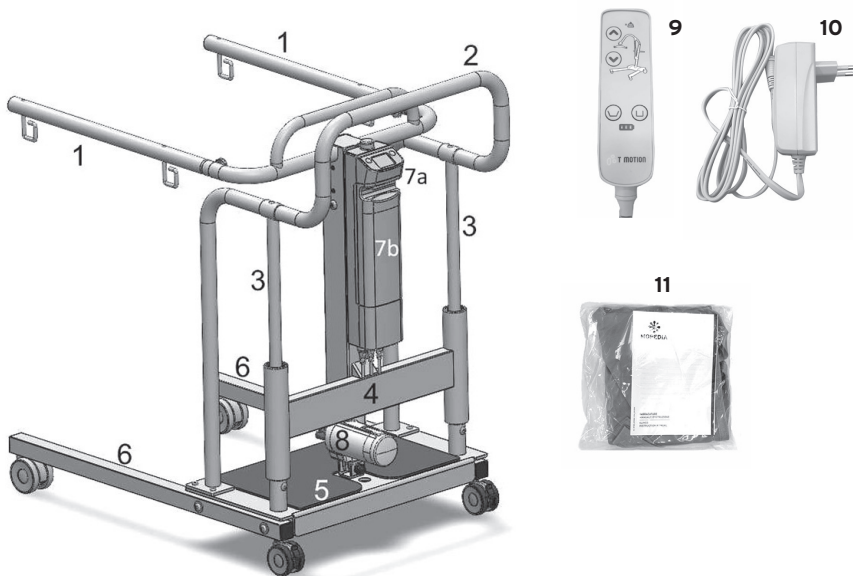
B Type

Max loading weight

## 7. GENERAL DESCRIPTION


### 7.1 Description of the parts of R1900

Images for illustrative purpose only

1. n.2 Removable arms equipped with attachment points for harness with locking handwheel
2. n.1 handle for movement
3. n.2 Sliding columns for lifting
4. n.1 Central body (sliding groups with bearings, support beam, support column, half-arms with handle for patient)
5. n.1 Basement equipped with:
  - n.1 Mat (basement upper part)
  - n.1 castor (basement lower part)
  - n.2 Actuators for leg opening-closing movement (basement lower part)
  - n.4 sliding groups for legs opening-closing movement (basement lower part)
6. n.2 Legs equipped with:
  - n.2 castors  $\varnothing$  75mm without brake (front part)
  - n.2 castors  $\varnothing$  65mm with brake (rear part)
7. n.1 Control unit (7a) with removable battery (7b)
8. n.1 Lifting actuator
9. n.1 Remote control
10. n.1 Battery charger
11. n.1 RIA200 standard harness (one size)

## 7.2 Description of the parts of RIA910 accessory arm

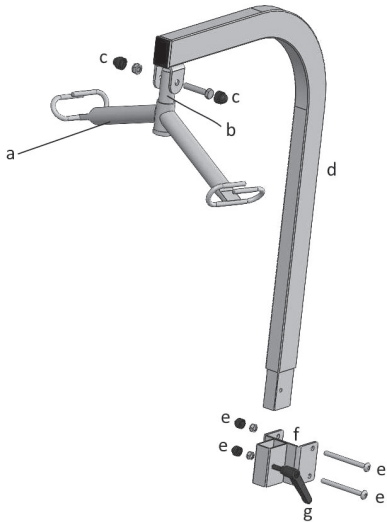
- a. 4 hooks slingbar
- b. slingbar support pin
- c. pin screw b. with nut and caps nut cover
- d. lifting arm
- e. screws with data and nut cover to install RIA910 accessory on RI900 central column
- f. plate to install the RIA910 accessory on central column RI900
- g. arm clamping handle

Image for illustrative purpose only

## 8. ASSEMBLY INSTRUCTIONS
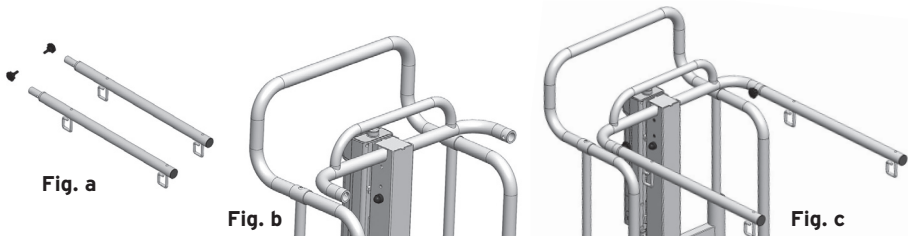
### 8.1 RI900 assembly instructions

1. Open the packing carton and check that the contents correspond to what indicated in the manual and that the parts have not been damaged during transport. In case otherwise, contact your dealer immediately.
2. Insert the patient support arms (fig. a) on the fixed part of the patient support handle (fig. b). Tighten the locking handwheels (fig. C) to secure the arms.

3. Install the battery in the appropriate housing of the control unit (fig. D). [Fig.d]

**Fig. d**

Images for illustrative purposes

The device is ready for use.

## 8.2 RIA910 accessory Assembly instructions

1. Open the packing carton and check that the contents correspond to what indicated in the manual and that the parts have not been damaged during transport. In case otherwise, contact your dealer immediately.
2. Remove the hole caps on the appropriate holes in the central column and install the plate on it (ref. f) using the appropriate screws (ref. e)
3. Assemble the lifting arm (ref. a, b, c)
4. Mount the lift arm on the special housing in the plate e tighten using the clamping handle (ref. g)

The accessory is ready for use.

Image for illustrative purpose

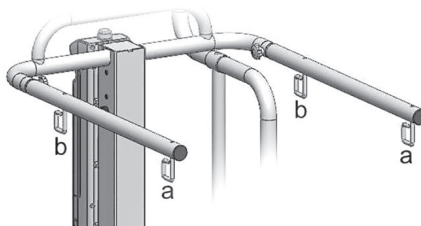
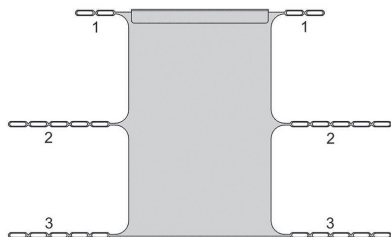
Note: for optimal use of the RIA910 lifting arm accessory, it is advisable to remove the RI900 lifting half-arms (ref. 1) by unscrewing the special tightening handwheels and pulling out the half-arms themselves; this will allow a better management of the lifted person by the operators, both for the operations to be performed on him and for his handling.

## 8.3 Assembly instructions for slings

The RIA200 and RIA210 harnesses are equipped with 6 attachment points (3 on each side) with adjustable heights through different eyelets corresponding to the different positions of the person's position.

Attach slings points 1 and 2 to hooks A

Attach slings points 3 to the hooks B

Images for illustrative purpose

## 9. INSTRUCTIONS FOR USE

### **WARNING!**

**ON FIRST USE IT IS RECOMMENDED TO CHARGE THE BATTERIES FOR ABOUT 6/8 HOURS. CONSEQUENTLY MAKE SURE THAT THE BATTERY IS CHARGED BEFORE EACH USE.**

### 9.1 Instructions for use of the R1900 patient lift

1. Approach the person to be lifted and position the lifter as close as possible to the person to be lifted, possibly opening the legs of the device if necessary
2. Before lifting, lock the rear wheel brakes
3. Keep the person centered between the legs of the base and always facing the caregiver
4. Carefully fasten the attachment points of the harness to the hooks on the lifting arms (see par. 8.3)
5. Operate the patient lifting actuator using the appropriate "▲" button on the control unit or on the remote control, lifting the patient to the necessary height, keeping him in the frontal position towards the caregiver.
6. Unlock the rear wheels and transfer the patient
7. Before starting the descent of the harnessed person, lock the rear wheels again
8. Activate the patient lowering actuator using the appropriate "▼" button on the control unit or on the remote control, lowering the patient to the necessary height.

NOTE: During normal operation of the lift it is possible to open or close the legs of the lift electrically using the special keys on the control unit and remote control dedicated to opening and closing the legs. The opening or closing operation legs can be performed both with the harnessed patient (with load) and without patient (without load).

#### WARNING!

ALWAYS CHECK THE CORRECT SITTING POSITION OF THE LIFTED PERSON DURING THE WHOLE MOVEMENT!

During the descent, avoid that the patient may encounter an obstacle that prevents the downward movement of the same. In that case, let the patient go up again and remove the cause of the impediment of descent.

### 9.2 Instructions for use of R1900 patient lift with RIA910 lifting arm

1. Approach the person to be lifted and position the lifter as close as possible to the person to be lifted, possibly opening the legs of the device if necessary
2. Before lifting, lock the rear wheel brakes
3. Keep the person centered between the legs of the base and always facing the caregiver
4. Carefully fasten the attachment points of the harness to the hooks on the lifting sling
5. Activate the patient lifting actuator using the appropriate "▲" button on the control unit or on the remote control, lifting the patient to the necessary height
6. Unlock the rear wheels and transfer the patient
7. Before starting the descent of the harnessed person, lock the rear wheels again
8. Activate the patient lowering actuator using the appropriate "▼" button on the control unit or on the remote control, lowering the patient to the necessary height.

NOTE: During normal operation of the lift it is possible to open or close the legs of the lift electrically using the special keys on the control unit and remote control dedicated to opening and closing the legs. The opening or closing operation legs can be performed both with the harnessed patient (with load) and without patient (without load).

#### WARNING!

ALWAYS CHECK THE CORRECT SITTING POSITION OF THE PERSON LIFTED DURING THE ENTIRE MOVEMENT!

During the descent, avoid that the patient may encounter an obstacle that prevents the downward movement of the same. In that case, let the patient go up again and remove the cause of the impediment of descent.

### 9.3 Feature Details

#### AUTOMATIC ANTI-CRUSHING FUNCTION

The electric kit features an automatic patient anti-crushing function which blocks the descent of the lifting arm in the event of accidental contact between the patient and an obstacle.

Image for illustrative purposes

#### EMERGENCY STOP

On the control unit there is a red "STOP" button (ref. 1) which, if pressed, immediately deactivates all commands. To restore operation, turn the red knob clockwise until it snaps up and returns to the rest position.

#### EMERGENCY COMMANDS


There are four buttons on the control unit (two for ascent / descent and two for opening / closing the legs) that can be used instead of those on the remote control to perform operations in the event that the remote shows some problems. operation, in particular:

Button 3: 'up' button for lifting

Button 4: leg opening button

Button 5: button to close the legs

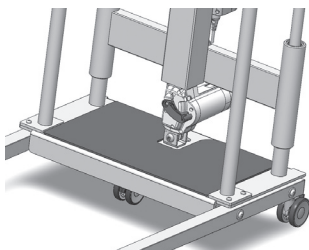Button 6: 'up' button for lifting

#### MECHANICAL LOWERING OF THE ACTUATOR

In case of use of the manual / mechanical descent, make sure that it is carried out in conditions of total safety for the patient. To activate the manual / mechanical descent, pull the red lever in the lower part of the actuator upwards and safely accompany the lowering of the device. Once lowering is complete, return the red lever to the rest position.

#### INDICATION OF THE BATTERY STATE OF CHARGE

There is a 3 LED indicator for the battery charge status. This indicator is located both on the control unit (ref. 2 in the photo above) and on the remote control (ref. 6 in the following photo). Three lit LEDs indicate a complete charge of the battery, while a single lit LED indicates that the battery is low and we need to recharge it.

#### HOW TO CHARGE THE BATTERIES

- Connect the adapter to the mains socket and then to the input of the control unit after removing the relative protective cap
- There is an orange LED on the charger adapter which is orange during the charging phase. Charge the battery until the LED turns green
- Before proceeding with the use of the lift remember to always disconnect the adapter to recharge the batteries!
- Charge the batteries for at least 6 hours before using the lift

#### 9.4 Instructions for using the remote control

The remote control is connected to the control unit via an extendable spiral cable.

**Buttons on the remote control:**

1. 'up' button for lifting
2. 'down' button for lowering
3. leg opening button
4. leg closing button

**Signaling on the Remote control:**

5. LED for overload indication: the lifter has an overload control device that stops movement if the excessive load threshold is exceeded and generates both a visual and acoustic signal
6. LED for signaling the state of charge of the battery

Image for illustrative purposes

**10. WARNINGS**

- The lift is a device that allows you to make short transfers of a patient with movement difficulties such as from a bed to a wheelchair or a comfortable chair or vice versa. The lift must not be used for patient transport.
- When lifting the person, make sure that the rear wheel brakes are locked.
- When transferring, avoid driving over obstacles that could cause you to tip over.

**WARNING!**

Always check the good condition of the harnesses.

Do not use slings that show signs of damage or wear.

The Muevo Home patient lift is not intended for use in direct contact with liquids, therefore pay particular attention to the protection of the on-board electronic equipment from jets of water or other liquids.

**11. CLEANING**

To clean the cradle and the structure use a solution of alcohol and neutral soap (in equal parts) mixed with water.

**12. MAINTENANCE**

- It is extremely important to check the lift before each use.
- More specifically, please check that all screws and bolts are well tightened.
- Please, check the lift can move freely with the wheels unlocked by the brakes.
- Check the rear wheel brakes function correctly.
- The mechanical parts must be lubricated with a drop of oil in order to help smoothness and wear resistance.
- Lubricate as above indicator every two or three months.

**13. SPARE PARTS AND ACCESSORIES**

For spare parts and accessories, please always refer to the MORETTI master catalogue.

Accessories

**Cod.RIA200** Standard harness (1 pcs supplied together with the RI900)

**Cod.RIA210** Harness for WC/toilet

**Cod.RIA910** Central lifting arm with 360° rotating 4 hooks sling bar

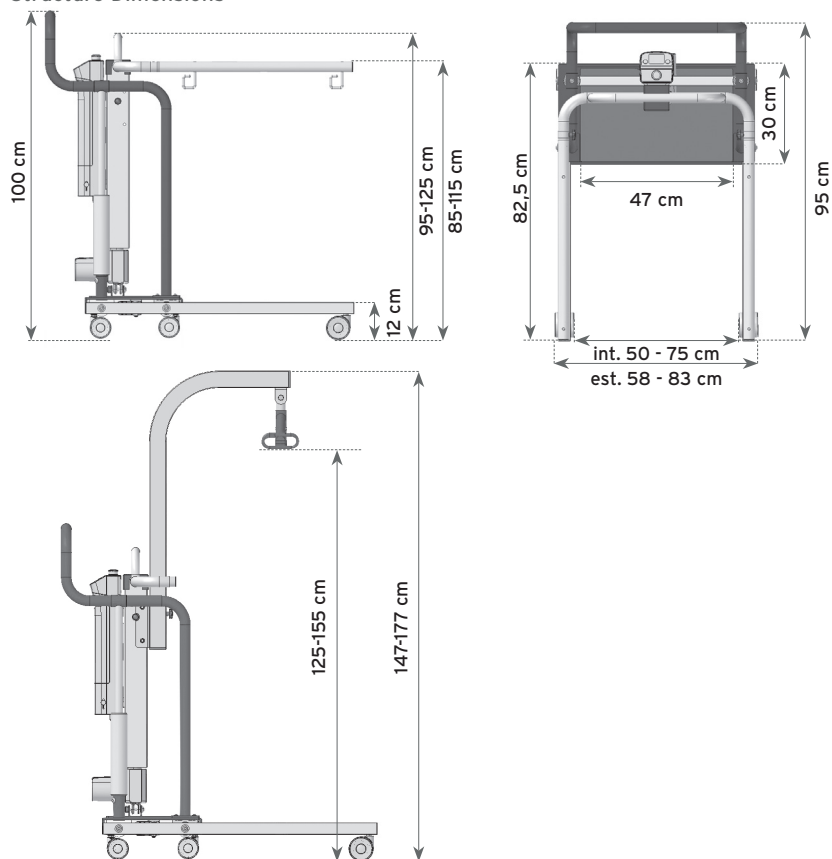
If you purchase the RIA910 accessory, you can use the following models of harnesses for MUEVO patient lifts:

- Cod.RP881 Standard harness with 'basic' headrest
- Cod.IM100-IM101 Standard harness without headrest
- Cod.IM110-IM111 Standard harness with headrest
- Cod.IM120 Containment harness with splints
- Cod.IM140 Toilet harness without headrest
- Cod.IM141 Toilet harness with headrest
- Cod.IM150 Containment harness for amputees

**ONLY USE ORIGINAL MORETTI ACCESSORIES. MORETTI S.P.A. COMPANY SHALL NOT BE LIABLE FOR ANY INCIDENT CAUSED BY IMPROPER USE.**

#### 14. TECHNICAL FEATURES

Structure Dimensions

With the supplied harnesses (RIA200, RIA210), it is possible to pick up a person starting from a height of about 25 cm from the ground.

Total weight with structure and electric kit: 59 Kg

Max capacity (patient weight): 135 Kg

## 15. CONDITIONS OF DISPOSAL

When disposing of the device never use the normal return systems of municipal solid waste. It is recommended instead to disposal through the joint municipal ecological islands for the expected operations of recycling the materials used.

### 15.1 Warnings for the proper disposal of the product, as required by the European Directive 2012/19/UE

At the end of its service life, the product must not be disposed of together with normal municipal waste. It must be delivered to the municipal sorted waste collection centre or to the resellers, in case they provide for this service. Sorted waste disposal reduces the environmental and health consequences of an improper disposal. Furthermore, the materials which make up the product may be recycled, thus resulting in energy and resource savings. In order to highlight the compulsory sorted disposal of the electro-medical equipment, the product itself bears the barred bin symbol.

### 15.2 Treatment of dead batteries (Directive 2006/66/CE)

The symbol on the product indicates that the batteries must not be treated like common household waste. Make sure the batteries are properly disposed; this prevents the possible environmental and health consequences of an improper disposal. Recycling helps to preserve natural resources. Deliver dead batteries to a proper recycling centre. For more detailed information regarding the disposal of dead batteries or of the product itself, please contact the municipality, the local waste disposal centre or the reseller from whom you purchased the device.

## 16. WARRANTY

Every Moretti product is guaranteed for 2 (two) YEARS from the purchase date, except the restrictions specified below. The warranty shall not apply in case of damage caused by improper use, abuse, alterations of the product or any other use not in compliance with the user manual. The proper use of the product is indicated in the user manual. Moretti S.p.A. shall not be liable for any damages or personal injuries due to incorrect assembly/improper use of the device, not compliant with the user manual. Moretti does not guarantee its products against damages or faults in the following cases: natural disasters, non authorized maintenance or reparations, damages caused by electrical supply problems (if provided), use of parts or components not provided by Moretti, failure in accomplishing the guidelines and instructions for use, non authorized alterations, damages due to the transport (different from Moretti original delivery) or failure in maintaining the device as indicated in the manual. The components subject to wear are not covered by this warranty if the damage is due to the normal use of the product.

### 16.1 Warranty for rechargeable batteries (if provided)

The original batteries and the spare batteries are covered by a (ninety) 90-days warranty as far as their performance is concerned and by a (six) 6-months warranty as far as manufacturing faults are concerned or as required by law. Warranty on fully charged batteries left in an unused state for more than (three) 3 consecutive months is automatically void. Warranty on fully discharged batteries left in an unused state for more than (three) 3 consecutive days is automatically void.

### 16.2 Reparations

If a Moretti product shows material or manufacturing faults during the warranty period, Moretti S.p.A., together with the client, will consider whether the product fault is covered by warranty. Moretti S.p.A., at its unquestionable discretion, may replace or repair the item under warranty in a Moretti reseller point or at its own qualified offices. Labour costs for repairing the product may be at Moretti's expense if it is determined the repair is under warranty. A repair or replacement does not renew nor extend the warranty.

A product, not covered by warranty, may be returned for reparation only after receiving prior authorization from Moretti client service. Labour and delivery costs for an out-of-warranty reparation will be fully at the client's or reseller's expense. Reparations of out-of-warranty products are guaranteed for 6 (six) months, from the day the client receives the repaired product.

The client is informed if, after examining and testing a returning product, Moretti decides the product is not defective. The product will be sent back to the client and all the return delivery costs will be at his expense.

### 16.3 Spare parts

The original Moretti spare parts are guaranteed for 6 (six) months from the day you receive them.

### 16.4 Exoneration clauses

Except what clearly specified in this warranty and in accordance with the law, Moretti S.p.A. does not offer its clients any other declaration, warranty or condition, expressed or implicit, including any declaration, warranty or condition of merchantability, of non violation or non interference, of suitability for a specific aim. Moretti S.p.A. does not guarantee that the use of Moretti products will be continuous and faultless. The duration of possible implicit warranties that may be applied in accordance with the law is limited to the warranty period. Some states or countries do not allow limitations to the duration of an implicit warranty, or the exoneration, or the limitations of accidental or indirect damages related to products for consumers. In such states or countries some exoneration or limitation of this warranty may not be applied to the user. This warranty is subject to modifications without any prior notice.

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