

PET CARRIER & CUSHION BLACK

DON'T THINK TWICE,
JUST BRING IT ALONG!



movinglife®

SEATING & SUPPORT



ATTO /ATTO
SPORT Armrests



Seat cushion



Seat cushion
leather



Luxury Swivel
Seat Leather



Seat & Cushion
Support Belt



Luxury Orthopedic
Backrest

STORAGE



Essentials
Pouch



Front & rear
basket



Caryall
& Cushion



Caryall
& Cushion leather



Pet Carrier &
Cushion Black



Flight Kit



Oxygen Bottle
Holder & Case

TECHNICAL



Cane / Crutches
Holders



Mobile Phone
Holder



Rear Light Lifting
Handle



ATTO Left Hand
Throttle Kit



Standard Flight
Battery



Launch - H2 2023

Long distance
battery - not flight
approved




Cable kit

ATTO Accessories are created specifically to enhance, support and work with the ATTO mobility scooters in all user modes; while driving, while wheeling along as trolley and when folded for storage or transportation.

SCAN TO LEARN
HOW TO COMBINE
OUR ACCESSORIES



How to Use PET CARRIER & CUSHION BLACK

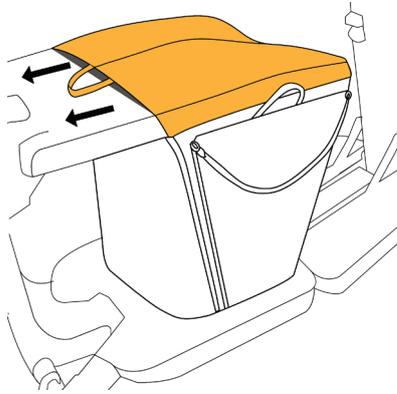

01


Slip the cushion over the seat (use handles to pull if needed)



02

Simply pull hook on elastic strap to fold under and connect to ATTO chassis

NOTE!
Take off before folding ATTO

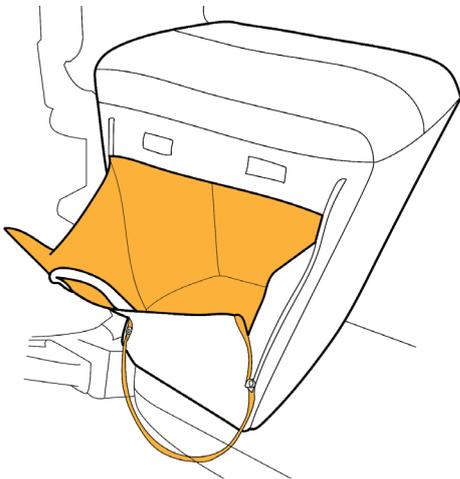

03

One-hand "pull to open" secure funnel access.
Quick "no zip" flap, to ensure your pet stays inside when closing.



04

Attach to ATTO Seat Cushion dedicated upper zip